



ZEUS SERIES

ELECTROSURGICAL UNIT



Utmost Safety and Reliability.
Thanks to an Advanced Safety Concept.



ARGON-Z

Feature

- Convenient LCD display on the front plate.
- Resection and coagulation by Argon Twin Button Handle.
- Foot switch is available for cutting and coagulation.
- 9 storages are available by using user storage function.
- Even coagulation by contactless coagulation.
- Small carbonization and almost none sticking.
- Fast hemostasis, when massive bleeding.
- Easy and clean range of vision due to small smoke.



ARGON-Z Specification

Rated Voltage	AC100V - AC240V	Minimum Purity of Argon	99.998%
Rated Frequency	50Hz / 60Hz	Argon Gas Flow, CUT	0.1 l/min ~ 9.9 l/min
Power Consumption	50VA + 10%	Argon Gas Flow, COAG.	0.1 l/min ~ 9.9 l/min
Fuse	T1A1 250V	Minimum Cylinder Pressure	200bar(20MPa)
Protection Class	Class 1, Type CF	Weight	6 Kg
Input Pressure	Min 2.5bar(250kpa) ~ Max 4.5 bar(450kpa)	Dimension(WxDxH)	330x380x130 mm

Standard Accessories

Code No.	Name of Item				
FS02-02R	Double Foot Switch				
CA05-01R	Foot Switch Signal Cable	FS02-02R	CA05-01R	CA06-01R	TB01-01R
CA06-01R	Monopolar Cable				
TB01-01R	Luer Lock Tube	CA07-02R	AP01-02R	CA03-01R	CA03-02R*2
CA07-02R	Reusable Flexible Argon Probe Cable				
AP01-02R	Reusable Flexible Argon Probe	RG01-01R			
CA03-01R	Ground Cable				
CA03-02R	Ground Cable (2ea)				
RG01-01R	Regulator				



- Main Frequency : 400, 500KHz
- Power Source : AC120V or AC230V, 50Hz / 60Hz
- Protection Class : Class 1, Type CF
- Dimension(WxDxHmm) : 388×410×160
- Weight : 12.5Kg

Feature

• Color Graphic User Interface

Wide TFT LCD mount, clean and sharper image quality, unlike the existing electricity and the innovative new U.I. configuration operation.

• Best Safety with Feedback System

REM(Return Electrode Monitoring) function monitors the size of contact area between a patient and the pad. If the size is inappropriate, it automatically blocks the high-frequency current to minimize the danger of burning incidents.

• All touch Screen System & key

Touch screen system through the implementation easier and faster with a parameter setting, external key management can be the same as a touch screen.

• Diversity of Feature Bipolar

Bipolar Cut, Bipolar Blend, Bipolar Forced to add features to implement more up-grade the optimal system.

Option 1 Endo Cut(Intra Cut)

Endo-cut is a monopolar electrosurgical treatment method and a cut which is carried out intervals(fractionated cut) with software-controlled alternations between cutting and coagulation stages.

Option 2 T.U.R.P.

Special resectoscopy, which removes the tissue in the prostate with kinetic plasma under saline liquid. (recommended to buy instruments for procedure by a user.)

Vessel Sealing

Seals vessels up to 7mm with bipolar coagulation during the procedure.(recommended to buy instrument for procedure by a user.)

* Option 2 function is included option 1 function.

Output Specification

	Mode	Output Power
CUT	Pure	400W / 200Ω
	Blend 1	250W / 200Ω
	Blend 2	200W / 200Ω
	Blend 3	150W / 200Ω
Coagulation	Contact	120W / 200Ω
	Spray	100W / 300Ω
	Argon	100W / 300Ω
Bipolar Coagulation	Standard	100W / 100Ω
	AutoStart	100W / 100Ω
	Forced	80W / 100Ω
Bipolar Cut	Cut	120W / 200Ω
	Blend	100W / 200Ω

Option 1

Endo Cut	Papillotomy	150W / 100Ω
	Polypectomy	150W / 100Ω
Coagulation	Soft	80W / 100Ω

Option 2

Bipolar TURP	TURP Cut	250W / 200Ω
	TURP Coag.	150W / 100Ω
Bipolar Coagulation	Vessel Sealing	250W / 100Ω

Standard Accessories

Code No.	Name of Item				
FS02-01R	Double Foot Switch				
FS01-03R	Single Foot Switch(ZEUS VISION) *Option 2: Not included	FS02-01R	* FS01-03R	HD02-01D	HD01-02R
HD02-01D	Disposable Twin Button Handle				
HD01-02R	Monopolar Handle & Cable	PL02-01R	PL03-02D	CA02-02R	CA03-01R
PL02-01R	Silicon Patient Plate(240X150mm)				
PL03-02D	Disposable Patient Plate (5ea)	CA01-02R	CA08-01R	BF01-01R	EL01-02D/EL02-02D
CA02-02R	Patient Return Plate Cable				EL02-02D/EL03-02D/EL04-02D
CA03-01R	Ground Cable				EL03-02D/EL04-02D/EL04-04D
CA01-02R	Bipolar Forceps Cable				
CA08-01R	Monopolar Connecting Cable(For Endo Cut*3Φ Connector)	CA01-02R	CA08-01R	BF01-01R	
BF01-01R	Bipolar Forceps(Bayonet 17.8cm 1.0 Tip)				
EL01-02D	Electrode(Knife 2.4*70mm)				
EL02-02D	Electrode(Ball 5mm)				
EL03-02D	Electrode(Loop 6mm)				
EL04-02D	Electrode(Needle 2.4*70mm)				
EL04-04D	Electrode(Needle Angled 2.4*70mm)				

ZEUS-400/300/200(ZEUS series **UPGRADED VERSION**)

The ZEUS series upgraded version provide safety flexibility and convenience by powerful and stable technology.

10 storages are available by using user storage function. Bipolar cut function is available.

- Main Frequency : 400, 500KHz
- Power Source : AC120V or AC230V, 50Hz / 60Hz
- Protection class : Class 1, Type CF
- Dimension(WxDxH mm) 330x455x150
- Weight : 13.0Kg

Application

- General Surgery
- Dermatology
- Vascular Surgery
- Gynecology
- ORL/ENT
- Pediatric Surgery
- Urology
- Orthopedics
- Oral Surgery
- Plastic Surgery
- Endoscopy

Feature

- A display window allows you to easily check output value, indicated as digits.
- Unit provides various functions that are required to perform an operation as functional cut mode (Pure cut, Blend1, Blend2, Blend3)Coagulation mode(Spray coag., Contact coag.), Bipolar coag. mode by hand or foot, Bipolar cut(ZEUS upgraded version only).
- Each working condition can be checked by sound and indication lamp.
- Hand bipolar(Auto bipolar)can be checked only by contacting Forceps.
- Unit output strong and stable power by micro-processor.
- Input power frequency is automatically detected.

Output Specification

Mode	ZEUS series UPGRADED VER.		
	ZEUS-400	ZEUS-300	ZEUS-200
Pure cut	400W / 300Ω	300W / 300Ω	200W / 300Ω
Blend 1	250W / 300Ω	230W / 300Ω	150W / 300Ω
Blend 2	200W / 300Ω	180W / 300Ω	120W / 300Ω
Blend 3	150W / 300Ω	120W / 300Ω	100W / 300Ω
Contact Coag.	120W / 300Ω	100W / 300Ω	100W / 300Ω
Spray Coag.	100W / 300Ω	80W / 300Ω	80W / 300Ω
Bipolar Coag.	100W / 100Ω	80W / 100Ω	80W / 100Ω
Bipolar Cut	100W / 200Ω	80W / 200Ω	80W / 200Ω

Standard Accessories

Code No.	Name of Item
FS02-01R	Double Foot Switch
FS01-02R	Single Foot Switch
HD02-01D	Disposable Twin Button Handle
HD01-01R	Monopolar Handle & Cable
PL02-04R	Silicon Patient Plate(240X150mm)
PL03-02D	Disposable Patient Plate (2ea)
CA02-03R	Patient Return Plate Cable
CA03-01R	Ground Cable
CA01-01R	Bipolar Forceps Cable
BF01-01R	Bipolar Forceps(Bayonet 17.8cm 1.0 Tip)
EL01-02D	Electrode(Knife 2.4*70mm)
EL02-02D	Electrode(Ball 5mm)
EL03-02D	Electrode(Loop 6mm)
EL04-02D	Electrode(Needle 2.4*70mm)
EL04-04D	Electrode(Needle Angled 2.4*70mm)

S-200	ZEUS-400	ZEUS-300	ZEUS-200
/ 300Ω	400W / 300Ω	300W / 300Ω	200W / 300Ω
/ 300Ω	250W / 300Ω	230W / 300Ω	150W / 300Ω
/ 300Ω	200W / 300Ω	180W / 300Ω	120W / 300Ω
/ 300Ω	150W / 300Ω	120W / 300Ω	100W / 300Ω
/ 300Ω	120W / 300Ω	100W / 300Ω	100W / 300Ω
/ 300Ω	100W / 300Ω	80W / 300Ω	80W / 300Ω
/ 100Ω	99W / 100Ω	80W / 100Ω	80W / 100Ω
/ 200Ω			



※ In case of Zeus-200, receptacle part is little different than Zeus-400/300, When customer buys Zeus-200, please check.

The ZEUS series provide safety flexibility and convenience by powerful and stable technology.

- Main Frequency : 400, 500KHz
- Power Source : AC120V or AC230V, 50Hz or 60Hz
- Protection class : Class 1, Type CF
- Dimension(WxDxH mm) 330x455x150
- Weight : 13.0Kg

Application

- General Surgery
- Dermatology
- Vascular Surgery
- Gynecology
- ORL/ENT
- Pediatric Surgery
- Urology
- Orthopedics
- Oral Surgery
- Plastic Surgery
- Endoscopy

Safety Function

- A fuse which is built in a power circuit board prevents over current from flowing through the equipment
- When the plate attached to the patient is disconnected from the unit, red alarm light begins flickering. If press any button of the twin button handle or press any pedal of double foot switch, alarm sound is made but the unit doesn't work.
- REM(Return Electrode Monitoring) function monitors the size of contacting area between a patient and the pad. If size is inappropriate, it automatically blocks the high-frequency current to minimize the danger of burning incident.
- To protect a patient, the case is fully grounded so that a leakage current can flow into the earth.



Output Specification

Mode	ZEUS-200S	ZEUS-150
Pure Cut	200W / 200Ω	150W / 200Ω
Blend Cut 1	150W / 200Ω	120W / 200Ω
Blend Cut 2	120W / 200Ω	100W / 200Ω
Contact Coagulation	80W / 200Ω	80W / 200Ω
Spray Coagulation	80W / 500Ω	60W / 500Ω
Bipolar Coagulation	80W / 100Ω	80W / 100Ω

- Main Frequency : 400, 500Khz
- Power Source : AC120V or AC230V, 50Hz / 60Hz
- Protection Class : Class 1, Type CF
- Dimension(WxDxHmm) : 330×330×150
- Weight : 8.0Kg

Feature

• Various Function with Compact Design

Cut(Pure Cut, Blend 1, Blend 2), Coag.(Contact, Spray), Bipolar Coag.: Fully enough for most of the operations.

• Best Safety with Feedback System

REM(Return Electrode Monitoring) function monitors size of contact area between a patient and the pad. If contact is inappropriate, it automatically blocks the high-frequency current to minimize the danger of burning incident.

• Different Tone Audible Sound

Each working condition can be checked by sound and indication lamp. The level if alarm sound can be adjusted during an operation of application mode.

• Auto Select Function

Without pressing cut or coag. button, use twin button handle or double foot switch for working cut or coag. mode. Press each mode button for 3 sec., output power will be set 1W, it enables to adjust from 1W to maximum power quickly.

Standard Accessories

Code No.	Name of Item				
FS02-01R	Double Foot Switch				
HD02-01D	Disposable Twin Button Handle				
HD01-01R	Monopolar Handle & Cable				
PL02-04R	Silicon Patient Plate(240X150mm)				
PL03-02D	Disposable Patient Plate (2ea)	FS02-01R	HD02-01D	HD01-01R	PL02-04R
CA02-03R	Patient Return Plate Cable				
CA01-01R	Bipolar Forceps Cable				
BF01-01R	Bipolar Forceps(Bayonet 17.8cm 1.0 Tip)				
EL01-02D	Electrode(Knife 2.4*70mm)				
EL02-02D	Electrode(Ball 5mm)	PL03-02D	CA02-03R	CA01-01R	BF01-01R
EL03-02D	Electrode(Loop 6mm)				
EL04-02D	Electrode(Needle 2.4*70mm)				
EL04-04D	Electrode(Needle Angled 2.4*70mm)				
		EL01-02D / EL02-02D		EL03-02D / EL04-02D / EL04-04D	

Electrosurgical Unit
ZEUS-100/80



Output Specification

Mode	ZEUS-100	ZEUS-80
Pure Cut	100W /300Ω	80W /300Ω
Blend Cut	100W /300Ω	80W /300Ω
Contact Coagulation	60W /300Ω	50W /300Ω
Bipolar Coagulation	80W /100Ω	70W /100Ω



- Main Frequency : 1.6MHz
- Power Source : AC120V or AC230V, 50Hz / 60Hz
- Protection Class : Class 1, Type BF
- Dimension(WxDxHmm) : 270×310×110
- Weight : 6.0Kg

Standard Accessories

Code No.	Name of Item			
FS01-02R	Single Foot Switch			
HD02-01D	Disposable Twin Button Handle			
PL02-05R	Silicon Patient Plate(165X90mm)			
EL01-02D	Electrode(Knife 2.4*70mm)			
EL02-02D	Electrode(Ball 5mm)			
EL04-02D	Electrode(Needle 2.4*70mm)			
EL04-04D	Electrode(Needle Angled 2.4*70mm)			
		EL01-02D / EL02-02D EL04-02D / EL04-04D		

Electrosurgical Unit
ZERO 50



Output Specification

Mode	ZERO50	
Cut	50W /300Ω	
Coagulation	40W /300Ω	



- Main Frequency : 1.6MHz
- Power Source : AC120V or AC230V, 50Hz / 60Hz
- Protection Class : Class 1, Type BF
- Dimension(WxDxHmm) : 200×220×90
- Weight : 2.6Kg

Standard Accessories

Code No.	Name of Item			
HD02-01D	Disposable Twin Button Handle			
PL02-03R	Silicon Patient Plate(110X80mm)			
		HD02-01D	PL02-03R	

Electrosurgical Unit

Accessories

Accessory	Code No.	Name of Item	Specification	Accessory	Code No.	Name of Item	Specification
	FS01-01R	Single Foot Switch (ZEUS-80)	3pin Plug Cable 4m		CA08-01R	Monopolar Connecting Cable	3Ø Connector
	FS01-02R	Single Foot Switch	3pin Plug Cable 4m		CA08-02R	Monopolar Connecting Cable	4Ø Connector
	FS01-03R	Single Foot Switch (ZEUS Vision)	3pin Plug Cable 4m		CA02-02R	Patient Return Plate Cable	6.3Pie Plug Silicon Cable 3m
	FS02-01R	Double Foot Switch	4pin Plug Cable 4m		CA02-03R	Patient Return Plate Cable	ValleyLab Type Plug Silicon Cable 3m
	PL01-01R	Reusable Patient Plate	Stainless Steel 6.3Pie Plug Cable 3m		CA03-01R	Ground Cable	Cable 4Ø, 3m
	PL02-01R	Silicon Patient Plate	240mm x 150mm 6.3Pie Plug Cable 3m		BF01-01R	Bipolar Forceps	Bayonet 17.8cm 1.0 Tip
	PL02-02R	Silicon Patient Plate	165mm x 90mm 6.3Pie Plug Cable 2m		BF01-02R	Bipolar Forceps	Bayonet 15.2cm 1.0 Tip
	PL02-03R	Silicon Patient Plate	110mm x 80mm 6.3Pie Plug Cable 2m		BF02-01R	Bipolar Forceps	Straight 9.0cm 0.5 Tip
	PL02-04R	Silicon Patient Plate	240mm x 150mm ValleyLab Type Plug Cable 3m		BF02-02R	Bipolar Forceps	Straight 12.1cm 1.0 Tip
	PL02-05R	Silicon Patient Plate	165mm x 90mm ValleyLab Type Plug Cable 2m		BF02-03R	Bipolar Forceps	Straight 17.8cm 1.0 Tip
	PL02-06R	Silicon Patient Plate	110mm x 80mm ValleyLab Type Plug Cable 2m		BF03-01R	Bipolar Forceps	Angled 9.0cm 0.5 Tip
	PL03-02D	Disposable Patient Plate	Adult Dual (120cm ²)		BF03-02R	Bipolar Forceps	Angled 17.8cm 1.0 Tip
	PL03-03D	Disposable Patient Plate	Pediatric Dual (66cm ²)		EL01-02D	Electrode	Knife 2.4 * 70mm Black
	HD01-01R	Monopolar Handle & Cable	Silicon Cable 4Ø 3m		EL02-02D	Electrode	Ball 5mm Black
	HD01-02R	Monopolar Handle & Cable	Silicon Cable 4Ø 3m		EL03-02D	Electrode	Loop 6mm Black
	HD02-01D	Disposable Twin Button Handle	3pin Plug Cable 3m		EL04-02D	Electrode	Needle 2.4 * 70mm Black
	CA01-01R	Bipolar Forcep Cable	2pin Plug Cable 3m EN Type Connector		EL04-04D	Electrode	Needle Angled 45° 2.4 * 70mm Black
	CA01-02R	Bipolar Forcep Cable	2pin Plug Silicon Cable 3m EN Type Connector		CT01-01R	CART	W390xD480xH850
	CA01-03R	Bipolar Forcep Cable	2pin Plug Silicon Cable 4m EU Type Connector		HD03-02R	Sens Probe	3 Pin Plug Cable 3m